AVA FERTILITY TRACKER

User Manual

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Not for contraceptive use

For additional information about your device and the Ava app visit www.avawomen.com/getstarted

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Ava cares about your privacy. All information you share is confidential and never shared with third parties.

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STEP 1

Download the Ava app from the App Store® or Google Play Store and create an account.

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STEP 2

Pair Ava with the app by plugging it in and following the onscreen instructions. Make sure the LED turns on.

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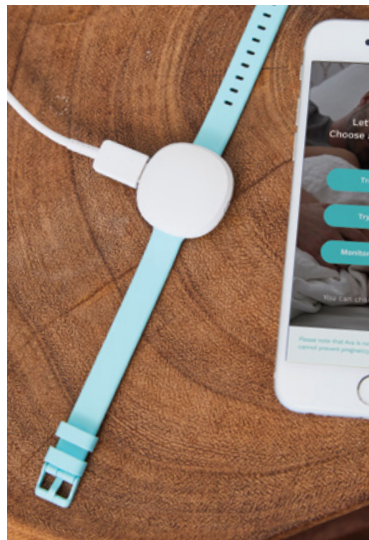
STEP 3

Charge Ava for at least two hours before your first use (until the LED turns solid white).

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STEP 4

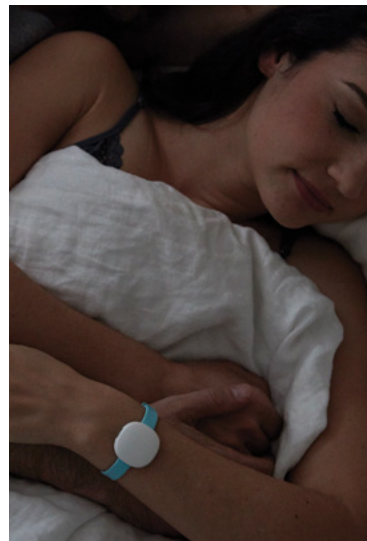
At night, when you put Ava on your wrist, it will vibrate to confirm it is ready to record.

- DE** Wenn du das Ava Armband abends am Handgelenk anlegst, vibriert es und bestätigt so, dass die Datenaufzeichnung beginnen kann.
- FR** La nuit, lorsque vous mettez Ava sur votre poignet, il vibre pour confirmer qu'il est prêt à enregistrer.
- ES** Cuando te pongas la pulsera Ava en tu muñeca por la noche, vibrará para confirmar que está lista para registrar datos.
- IT** La sera, quando indossi Ava, il dispositivo vibrerà per confermare che è pronto a registrare dati.

STEP 5

Wear Ava snugly during the night for at least four hours. Start on any day of your cycle.

- DE** Ava sollte während der Nacht für mindestens vier Stunden eng an Deinem Handgelenk anliegen. Du kannst an jedem Tag Deines Zyklus beginnen.
- FR** Portez Ava confortablement pendant au moins quatre heures par nuit. Commencez quel que soit le jour de votre cycle.
- ES** Ajusta tu Ava correcta y cómodamente por la noche durante al menos cuatro horas. Empieza durante cualquier día de tu ciclo.
- IT** Indossa Ava comodamente durante la notte per almeno quattro ore. Puoi incominciare in qualunque giorno della settimana.

STEP 6

Sync your data every morning by opening the app and tapping the sync button on the home screen.

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STEP 7

Wear Ava every night. Ava becomes more accurate the more cycles you record.

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The Ava LED indicator

- Solid pink = Ava needs to be synced
- Flashing pink = Ava needs to be charged
- Flashing white = Charging in progress
- Solid white = Charging complete
- Solid white & vibration = Ava is ready to record data
- Flashing blue = Data is transmitted

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Please read the following directions and safety information carefully before you use Ava.

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Safety related symbols indicate all important instructions for safe use of tdevice. Ignoring these indications can lead to serious damage on the device or safety exposure for the user.

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Representative in the USA

Ava Science, Inc.
77 Geary Street
San Francisco, CA 94108, USA

For more information go to: www.avawomen.com

Packaging liner PNA 10002 / Version 1.5
Printed in China.

© 2018 Ava AG

EN – DIRECTIONS & SAFETY INFORMATION

INDICATIONS FOR USE

The Ava Fertility Tracker is intended to monitor a woman's fertility by measuring and recording physiological parameters (body temperature, resting pulse rate, heart rate variability, and breathing rate) as an aid in ovulation prediction to aid in conception (not to be used for contraception).

INTENDED USE

The Ava Fertility Tracker is a non-invasive device intended to measure and display physiological parameters to aid women in ovulation prediction to facilitate conception. Additionally, parameters are collected to improve the quality of the prediction, and to provide general information on health and wellness.

Temperature is the most important measurement in support of the Ava Fertility Tracker's prediction of the fertile window. This, in combination with historical menstrual cycle data, is the basis for the Fertility Tracker technology and the intended use of the device. The Ava device (bracelet) is combined with smartphone applications to display the physiological parameters and to gather user inputs.

CONTRAINDICATIONS

Ava is not a contraceptive and cannot be used to prevent pregnancy.

PRECAUTIONS & WARNINGS

Not for cycles outside of 24 – 35 days range.

Not for women with infertility issues.

Contact your physician if you have questions on getting pregnant or if you are not pregnant after six months of actively trying to conceive.

Do not use if you suffer from cholinergic urticaria (sweat induced allergies).

If you have tendinitis, carpal tunnel syndrome, or other musculoskeletal disorders, consult your doctor prior to use.

Keep this product away from other high frequency devices which may interfere with operation.

The Ava device is not to be used for fever measurement and to diagnose, treat, cure, or prevent any disease.

Please carefully read the instructions for use prior to wearing the device.

HANDLING, OPERATION & STORAGE

The Ava device is not waterproof. Avoid contact with any liquid. Do not operate in environment with more than 80% relative humidity.

Do not use the Ava device in a sauna or steam room. The maximum ambient temperature for operation is 38°C / 100°F.

Do not use the Ava device under influence of low atmospheric pressure.

Do not operate when pressure is below 740 mbar.

Use skin care products sparingly on the areas of your skin covered by the Ava device.

Do not wear your device while charging it. Do not charge your device while it is wet.

The device can heat up during charging. Do not use tablet chargers or fast chargers. Only charge your device using a computer, powered hub or po-wer supply that is certified by a recognized testing laboratory.

Do not place on flammable surfaces when charging.

Do not use a damaged USB cable to charge your device.

Do not drop, knock, or shake the device. Rough handling and vibration can damage internal circuit boards and sensitive mechanical components.

The use and standby times are approximate, and may be affected by device settings, battery condition, temperature and other factors.

Do not apply mechanical stress.

Do not open the enclosure or disassemble the device. Do not attempt to remove the battery from the device.

Substances in this product and its battery may harm the environment or cause injury if handled and disposed improperly. Do not dispose of the device in a fire. The battery could explode.

Do not store or transport the device in hot conditions (above 35°C / 95°F) or cold conditions (below 5°C / 41°F).

Do not store or transport the device in moist conditions (no more than 80% relative humidity)

Do not store or transport the device under influence of low atmospheric pressure (not lower than 740 mbar).

Do not use harsh chemicals, cleaning solvents or powerful detergents to clean the device. Use a clean, damp cloth. Do not wash the Ava device under a faucet.

Avoid static electricity, ESD and high voltage as these may damage the product.

Do not expose the product to the following conditions: Corrosive gasses such as Cl₂, H₂S, NH₃, SO₂, or NO_x Extreme humidity or salty air.

Prolonged exposure to direct sunlight. Temperatures

beyond those specified for operation.

This package contains small parts that may be hazardous to children and should be kept out of reach of small children. This product is not a toy. Do not allow children or pets to play with the device.

ADVERSE EVENTS

The device uses biocompatible materials. Wearing the device snugly for extended time can lead to skin irritation or sensitization due to rubbing and friction. Wear the device only during the night to allow your skin to dry and breathe during the day.

If signs of an allergic reaction occur or local irritation persists, please discontinue use and contact your physician.

Remove your device if it feels hot.

If you feel soreness, tingling, numbness, burning or stiffness in your hands or wrists while or after wearing the product, please discontinue use.

AVA LIMITED PRODUCT WARRANTY

This Limited Warranty Policy ("Warranty") is offered by Ava AG, a company formed under the laws of Switzerland with domicile in Zurich, Switzerland, and its subsidiary Ava Science, Inc., a Delaware corporation with principal business in San Francisco, California (collectively "Ava" or "we"). The Warranty applies to the sale and use of Ava-branded fertility tracking bracelet marked with the "Ava" trade name or logo and containing a valid Ava serial number which were sold/purchased on Ava's website ("Site") or from an authorized reseller ("Product").

This Warranty is also governed by the most current version of the Terms of Service ("Terms"), which are located at www.avawomen.com/terms-of-service and are hereby incorporated into this Warranty by reference as though set forth in full herein. In case of a conflict between the Terms and the Warranty, the Terms shall prevail.

BY USING YOUR PRODUCT, YOU ARE AGREEING TO BE BOUND BY THE TERMS OF THIS WARRANTY. DO NOT USE YOUR PRODUCT UNTIL YOU HAVE READ THE TERMS OF THE WARRANTY. IF YOU DO NOT AGREE TO THE TERMS OF THE WARRANTY, DO NOT USE THE PRODUCT.

1. Scope of Warranty.

This Warranty applies only to the original purchaser ("you") who purchased the Product from an authorized source.

This Warranty gives you specific legal rights, and you may have other legal rights, including under consumer laws, that vary in different jurisdictions. To the extent permit-

ted by law, this Warranty and the remedies set forth are exclusive and in lieu of all other warranties, remedies and conditions, whether oral, written, statutory, express or implied. Ava disclaims all statutory and implied warranties, including without limitation, warranties of merchantability and fitness for a specific purpose and warranties against hidden or latent defects, to the extent permitted by law. In so far as such warranties cannot be disclaimed, Ava limits the duration and remedies of such warranties to the duration of the Warranty (to the extent permitted by law) and, at Ava's option, the repair or replacement services described below. Some jurisdictions do not allow limitations on how long an implied warranty (or condition) may last, so the limitation described above may not apply to you.

2. Product Warranty.

Ava warrants that the Product will be free from material defects in workmanship and materials under normal use for a period of one (1) year from the date of your purchase, unless longer minimum periods are required under mandatory laws in your jurisdiction ("Warranty Period").

3. Remedies.

If you submit warranty claims to Ava before the end of the Warranty Period and your Product clearly and verifiably fails the warranty, Ava will, in its sole discretion: (i) refund you in the amount of the price you paid for the Product (excluding taxes and shipping costs); (ii) exchange the Product with a new or refurbished Product (exchange the Product of identical specifications or functional equivalent); or (iii) repair the Product. This Section sets forth your sole and exclusive remedy, to the extent permitted by law. Returns shipped without Ava's prior express return authorization will not be accepted.

If a Product has been exchanged or repaired following this Warranty, it will be subject to a warranty for either: (i) the remaining original Warranty Period; or (ii) thirty (30) days from the date of repair or replacement or for any additional period that is applicable in your jurisdiction. This Warranty does not cover the software embedded in the Product and the services provided by Ava to owners of the Product, nor your use of the Site and the application provided by Ava. Please refer to the Terms for details of your rights with respect to their use.

4. Obtaining Warranty Service.

You must deliver the Product to the address specified by Ava in either its original packaging or packaging providing an equal degree of protection. In accordance with applicable law, Ava may require that you furnish proof of purchase details and/or comply with registration require-

ments before any warranty service is rendered. It is your responsibility to synchronize any data stored on the Product. It is likely that such data will be lost or reformatted during service, and Ava will not be responsible for any such damage or loss. Ava is not responsible for lost or damaged return shipments. You assume responsibility for the payment of any shipping charges and for all risk of loss or damage to the Product during shipment to Ava.

5. Exclusions and Limitations.

This Warranty does not apply to, and explicitly excludes, the following: (a) consumable parts, such as straps, unless damage has occurred due to a defect in materials or workmanship in the Product; (b) normal wear and tear and cosmetic damage, including, but not limited to, scratches, dents, damaged packaging material or damaged plastic parts; (c) defects or damage caused by alteration, unusual stress, accident (including, without limitation, collision, modification fire and the spillage of food or liquid), neglect, abuse, misuse, unauthorized repair, installation, wiring, improper storage or testing, use of third party accessories such as straps or cables, third-party applications downloaded to the Product or if the serial number has been removed; (d) damage caused by using the Product outside the permitted or intended uses described by Ava; (e) damage caused by use with non-Ava products; (f) any non-Ava hardware products or any software, even if packaged or sold with the Product; and (g) modifications of the Product by anyone other than Ava. Ava reserves the right to disable Product if Ava suspects fraud or if Ava finds out that the Product is counterfeit, stolen or lost in shipment. This Warranty does not cover software embedded in the Product and the services provided by Ava to owners of the Product. Please refer to the Terms for details of your rights with respect to their use.

6. Implied Warranties.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE MANDATORY LAW, ALL IMPLIED WARRANTIES (INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A SPECIFIC PURPOSE) SHALL BE EXCLUDED OR, IF LEGALLY NOT PERMITTED, LIMITED IN DURATION TO THE DURATION OF THIS LIMITED WARRANTY.

Some jurisdictions do not allow limitations on the duration of an implied warranty, so the above limitation may not apply to you.

7. Limitation of Damages.

Ava SHALL NOT BE LIABLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES WHETHER ARISING IN AN ACTION (INCLUDING WITHOUT

LIMITATION LOSS OF REVENUE, USE, PROFITS, DATA, OR GOODWILL OR COSTS OF PROCURING SUBSTITUTE PRODUCTS) FOR BREACH OF CONTRACT, TORT, PRODUCT LIABILITY OR ANY OTHER CAUSE OR THEORY OF LIABILITY, RELATED TO OR IN CONNECTION WITH THIS WARRANTY, THE TERMS OR THE PURCHASE, SALE, USE, OPERATION OR PERFORMANCE OF THE PRODUCTS. AVA AND YOU HAVE AGREED THAT THESE LIMITATIONS WILL SURVIVE AND APPLY EVEN IF ANY LIMITED REMEDY SPECIFIED IN THESE TERMS IS FOUND TO HAVE FAILED OF ITS ESSENTIAL PURPOSE. YOU AGREE AND ACKNOWLEDGE THAT THE TOTAL AMOUNT OF AVA'S LIABILITY, IF ANY, FOR ANY AND ALL CLAIMS, CAUSES OF ACTION, LOSSES, DAMAGES, OR JUDGMENTS ARISING OUT OF OR RELATED TO THIS AGREEMENT OR THE SERVICES SHALL NOT EXCEED THE COST OF THE PRODUCT AT ISSUE OR \$500.00, WHICHEVER IS GREATER.

Some jurisdictions do not allow the limitation or exclusion of liability for incidental or consequential damages, so the above limitation or exclusion may not apply to you.

©2018 AVA AG, all rights reserved. Ava and Ava logo type are trademarks of Ava AG. Bluetooth is a trademark of Bluetooth SIG Inc. iPhone and iOS are trademarks of Apple Inc., registered in the US and other countries. Android is a trademark of Google Inc. All other trademarks that are not mentioned by name are property of their respective owners.

Patent pending. Swiss design and technology.

Made in Europe.

For more information visit www.avawomen.com

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USA

FCC ID: 2APV7-AVA02

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference.
- (2) this device must accept any interference received, including interference that may cause undesired operation.

The Ava Fertility Tracker has been designed and complies with the safety requirements for portable RF exposure in accordance with FCC rule part §2.1093 and KDB 447498 D01.

Canada

IC: 23955-AVA02 Model 2

EN: This device complies with ISSED's license-exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device

This device complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 for portable use conditions.

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Europe (EU/EFTA)

EN: Declaration of conformity with regard to the EU Directive 2014/53/EU.

Ava AG is authorized to apply the CE mark on its product, thereby declaring conformity to the essential requirements and other relevant provisions of Directive 2014/53/EU and other applicable directives. The complete declaration of conformity can be found in the specific product section at www.avawomen.com/quality

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ABBREVIATIONS

ISED: Innovation, Science and Economic Development Canada

FCC: Federal Communications Commission

ESD: Electrostatic Discharge / Descargas Electrostáticas

PCOS: Polycystic Ovary Syndrome / Polyzystisches Ovar Syndrom / Síndrome dell'Ovaio Policistico

www.avawomen.com/getstarted

www.avawomen.com/de/esgehtlos

www.avawomen.com/fr/commencer

www.avawomen.com/es/empieza

www.avawomen.com/it/siinizia